



Lymphoedema
Supply Company

VARISAN TOP

LYMPHOLOGY, PHLEBOLOGY
AND VASCULAR LINE

BY CIZETA MEDICALI



CIZETA MEDICALI produces therapeutic and preventive elastic compression products that protect the upper and lower limbs against venous and lymphatic disorders.

All products are made using innovative technology and certified on an international level.



VARISAN TOP LYMPHOLOGY, PHLEBOLOGY AND VASCULAR TOP LINE

Elastic compression therapy remains the best known and most effective form of treatment that adapts best to various expressions of venous and lymphatic disease.

The application of external compression counteracts the effects of blood and lymph stasis.

The following compression stockings, glove and arm sleeves are available in several lengths, degrees of compression and materials; each characteristic should be chosen on the basis of the type of disease to be treated.

- Medical compression stocking
- Therapeutic arm sleeve
- Therapeutic glove

THERAPEUTIC MEDICAL COMPRESSION STOCKING

CHARACTERISTICS

The VARISAN® TOP line is RAL certified and available in 5 sizes and 2 lengths. The product is very comfortable as it is made of a multifilament yarn, which makes the stocking soft, easier to put on and allows it to breathe. Many products are available with cotton in contact with the skin. All the articles in this line can also be made to measure.

INDICATIONS

CLASS I (18 - 21 mmHg) standard and made to measure

Varices, varices during and after pregnancy, initial venous oedema, reticular veins.

CLASS II (23 - 32 mmHg) standard and made to measure

Superficial vein thrombosis, acute deep vein thrombosis, lymphoedema, prevention of post-thrombotic syndrome.

CLASS III (34 - 46 mmHg) standard and made to measure

- valvular insufficiency of the deep veins, prevention of ulcer relapse, lymphoedema, post-thrombotic syndrome with severe skin tissue impairment.
- primary and secondary lymphoedema at various clinical stages
- post-traumatic or post-operative oedema
- lymphoedema in its various stages
- phlebolymphoedema
- keloid lymphoedema following surgery

CONTRAINDICATIONS

Acute dermohypodermatitis, skin infections, peripheral obliterant arteriopathy, known contact allergy to elastic support appliances.

DURATION OF TREATMENT

Compression therapy with VARISAN® medical compression stockings should only be given during the daytime. The stocking is to be put on in the morning, if possible before getting up, and taken off in the evening, before going to bed. During rest, the compression exerted could prove to be excessive. For short afternoon rests there is no need to remove the stocking. It is recommended to walk as much as possible during the daytime.



Colours

739 beige



VARISAN® TOP

Medical Compression Stocking

GUARANTEE


The high quality of the materials used to make the VARISAN® elastic compression support appliance and the technologies adopted in their production guarantee the product for 6 months starting from the first time it is worn, providing the user instructions are followed.









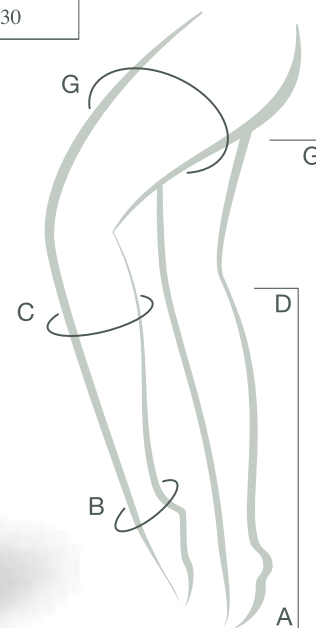

Size		CIRCUMFERENCE (cm)									
		1		2		3		4		5	
		NORMAL	PLUS	NORMAL	PLUS	NORMAL	PLUS	NORMAL	PLUS	NORMAL	PLUS
Thigh	G	50-55	53-58	53-58	56-61	56-61	60-66	60-66	64-70	64-70	70-77
Calf	C	31-34	34-37	34-37	37-40	37-40	40-44	40-44	44-48	44-48	48-52
Ankle	B	19-20		21-23		23-25		25-27		28-30	

LENGTH (cm)			
		Short	Normal
Knee-high stocking	AD	36 - 41	42 - 48
Full-length stocking	AG	66 - 73	74 - 82

also available made to measure

Compression classes available	
Class	Compression in mmHg
I	18 - 21
II	23 - 32
III	34 - 46



COMPRESSION CLASS				
REF.	18 - 21 mmHg	23 - 32 mmHg	34 - 46 mmHg	ARTICLE
Closed toe 	623E2			Right single-leg tights (AGTR), short
	623N2			Right single-leg tights (AGTR), normal
	623E3			Left single-leg tights (AGTL), short
	623N3			Left single-leg tights (AGTL), normal
Open toe 	6A3E2	6A4E2	6A5E2	Right single-leg tights (AGTR), short
	6A3N2	6A4N2	6A5N2	Right single-leg tights (AGTR), normal
	6A3E3	6A4E3	6A5E3	Left single-leg tights (AGTL), short
	6A3N3	6A4N3	6A5N3	Left single-leg tights (AGTL), normal
	CA3E2	CA4E2		Right single-leg tights (AGTR), short
	CA3N2	CA4N2		Right single-leg tights (AGTR), normal
	CA3E3	CA4E3		Left single-leg tights (AGTL), short
	CA3N3	CA4N3		Left single-leg tights (AGTL), normal
Closed toe 	6235E	6245E		Tights (AT) short
	6235N	6245N		Tights (AT) normal
Open toe 	CA3E5	CA4E5		Tights (AT) short
	CA3N5	CA4N5		Tights (AT) normal
Closed toe 	623E1	624E1		Knee-high stocking (AD), short
	623N1	624N1		Knee-high stocking (AD), normal
Open toe 	6A3E1	6A4E1	6A5E1	Knee-high stocking (AD), short
	6A3N1	6A4N1	6A5N1	Knee-high stocking (AD), normal
	CA3E1	CA4E1		Knee-high stocking (AD), short
	CA3N1	CA4N1		Knee-high stocking (AD), normal
Closed toe 	623E4			Full-length stocking (AG), short
	623N4			Full-length stocking (AG), normal
Open toe 	6A3E4	6A4E4	6A5E4	Full-length stocking (AG), short
	6A3N4	6A4N4	6A5N4	Full-length stocking (AG), normal
	CA3E4	CA4E4		Full-length stocking (AG), short
	CA3N4	CA4N4		Full-length stocking (AG), normal
Closed toe 	623E9	624E9		Self-supporting stocking (AD), short
	623N9	624N9		Self-supporting stocking (AD), normal
Open toe 		6A4E9	6A5E9	Self-supporting stocking (AD), short
		6A4N9	6A5N9	Self-supporting stocking (AD), normal

Products with cotton in contact with the skin

“

very comfortable as it is made of a multifilament yarn, which makes the stocking soft, easier to put on and allows it to breathe



THERAPEUTIC ARM SLEEVE

CHARACTERISTICS

The VARISAN® TOP arm sleeve line is available in 3 sizes and 2 lengths. All the articles in this line are also available made to measure.

The upper hem is coated with silicone to ensure it adheres to the limb. All products have cotton in contact with the skin. Also available in a version with a support belt

INDICATIONS

CLASS II (23 - 32 mmHg) standard and made to measure

Treatment of lymphoedema of the upper limb, post-thrombotic syndrome, maintenance of the results obtained with adequate treatment of the lymphoedema.

CONTRAINDICATIONS

Acute lymphangitis or erysipelas, secreting skin lesions, infections, known contact allergy to elastic support appliances.

DURATION OF TREATMENT

Compression therapy should only be given during the daytime. The sleeve is to be put on in the morning, if possible before getting up and taken off in the evening. It is recommended to exercise the limb during the daytime.



Compression classes available

Class	Compression in mmHg
II	23 - 32

Colours

739 beige




VARISAN® TOP

Therapeutic Arm Sleeve

CIRCUMFERENCE (cm)				
Size		S	M	L
Arm	F	25 - 29	29 - 33	33 - 38
Forearm	D	21 - 25	25 - 29	29 - 33
Wrist	C	15 - 18	18 - 21	21 - 24

LENGTH (cm)		
	Short	Long
GC	< 45	> 45

also available made to measure

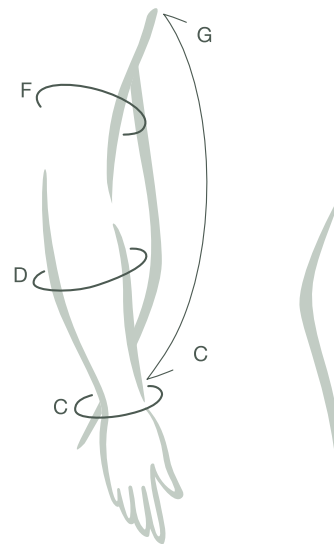

COMPRESSION CLASS		
REF.	23 - 32 mmHg	ARTICLE
	6C444	Short arm sleeve to wrist
	6L444	Long arm sleeve to wrist
	6C445	Short arm sleeve to metacarpus with thumb
	6L445	Short arm sleeve to metacarpus with thumb
	6C449	Short self-supporting arm sleeve to metacarpus
	6L449	Long self-supporting arm sleeve to metacarpus

Products with cotton in contact with the skin



THERAPEUTIC GLOVE

CHARACTERISTICS

The VARISAN® TOP glove line is available in 3 sizes. All products have cotton in contact with the skin. All the articles in this line are also available made to measure.

INDICATIONS

CLASS II (23 - 32 mmHg) standard and made to measure.

Treatment of lymphoedema of the upper limb, maintenance of the results obtained through adequate treatment of lymphoedema of the arm and hand.

CONTRAINDICATIONS

Acute lymphangitis or erysipelas, secreting skin lesions, infections, known contact allergy to elastic support appliances.

DURATION OF TREATMENT

Compression therapy should only be given during the daytime. The sleeve is to be put on in the morning before getting up and taken off in the evening. It is recommended to exercise the limb during the daytime.



Compression classes available

Class	Compression in mmHg
II	23 - 32

Colours

739 beige




VARISAN® TOP

Therapeutic Glove


CIRCUMFERENCE (cm)			
Size	S	M	L
A	18 - 20	20 - 22	22 - 24
B	19 - 21	21 - 23	23 - 25

LENGTH (cm)	
AB	3 - 5 cm
AC	9 - 12 cm

also available made to measure

COMPRESSION CLASS		
REF.	23 - 32 mmHg	ARTICLE
	6A443	Multistretch cotton glove

Products with cotton in contact with the skin

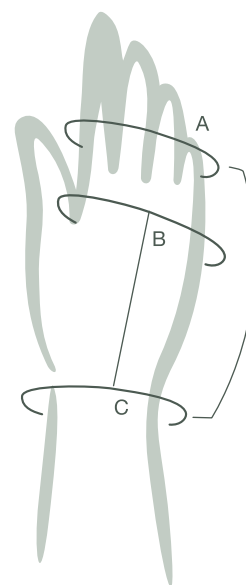



TELEPHONE

+61 438 806 337

EMAIL

sales@ecsconsulting.com.au

WEB

lymphoedemasc.com.au