



Lymphoedema  
Supply Company

# VARISAN FASHION

PHLEBOLOGY LINE  
BY CIZETA MEDICALI



CIZETA MEDICALI produces therapeutic and preventive elastic compression products that protect the upper and lower limbs against venous and lymphatic disorders.

All products are made using innovative technology and certified on an international level.

# VARISAN FASHION PHLEBOLOGY LINE

Elastic compression therapy remains the best known, widely experimented and effective therapeutic tool at our disposal and is easily adaptable to the various expressions of venous disease. It is the only form of therapy capable of constantly counteracting the state of venous hypertension that causes tissue damage.

The following ranges of medical compression stockings are available in several lengths, degrees of compression and colours; each characteristic should be chosen on the basis of the type of disease to be treated.

- Fashion
- Fashion Innova

# FASHION

## CHARACTERISTICS

The VARISAN® FASHION line is RAL certified and available in 5 sizes and 2 lengths.

The product is very comfortable as it is made of a multifilament yarn, which makes the stocking soft, easier to put on and allows it to breathe.

Its variety of colours encourages greater use in day-to-day life.

## INDICATIONS

CLASS I (18 - 21 mmHg)

- Varices, varices during and after pregnancy, initial venous oedema, reticular veins

CLASS II (23 - 32 mmHg)

- Superficial vein thrombosis, acute deep vein thrombosis, lymphoedema, prevention of post-thrombotic syndrome.

## CONTRAINDICATIONS

Acute dermohypodermatitis, skin infections, peripheral obliterant arteriopathy, known contact allergy to elastic support appliances.

## DURATION OF TREATMENT

Compression therapy with VARISAN® medical compression stocking should only be given during the daytime. The stocking is to be put on in the morning, if possible before getting up and taken off in the evening, before going to bed. During rest, the compression exerted could prove to be excessive. For short afternoon rests there is no need to remove the stocking.

We recommend as much walking as possible during the daytime.

## GUARANTEE

The high quality of the materials used to make the VARISAN® elastic compression support appliance and the technologies adopted in their production guarantee the product for 6 months starting from the first time it is worn, providing the user instructions are followed.



Compression classes available	
Class	Compression in mmHg
I	18 - 21
II	23 - 32









Colours	
620 llama	
720 coffee	
862 black	



# VARISAN® FASHION

## Phlebology Line Medical Compression Stockings

CIRCUMFERENCE (cm)						
Size	1	2	3	4	5	
Thigh	<b>G</b>	49 - 56	54 - 62	59 - 67	64 - 74	72 - 82
Calf	<b>C</b>	29 - 35	33 - 40	38 - 45	42 - 48	45 - 52
Ankle	<b>B</b>	19 - 21	22 - 24	25 - 27	28 - 30	31 - 34

LENGTH (cm)			
		SHORT	NORMAL
Knee-high stocking	<b>AD</b>	36 - 41	42 - 48
Full-length stocking	<b>AG</b>	66 - 73	74 - 82

COMPRESSION CLASS			
REF.	18 - 21 mmHg	23 - 32 mmHg	ARTICLE
	F23E1	F24E1	Closed toe short knee-high stocking (AD)
	F23N1	F24N1	Closed toe normal knee-high stocking (AD)
	F23E9	F24E9	Closed toe short self-supporting stocking (AG)
	F23N9	F24N9	Closed toe normal self-supporting stocking (AG)
	F23E5	F24E5	Closed toe short tights (AT)
	F23N5	F24N5	Closed toe normal tights (AT)

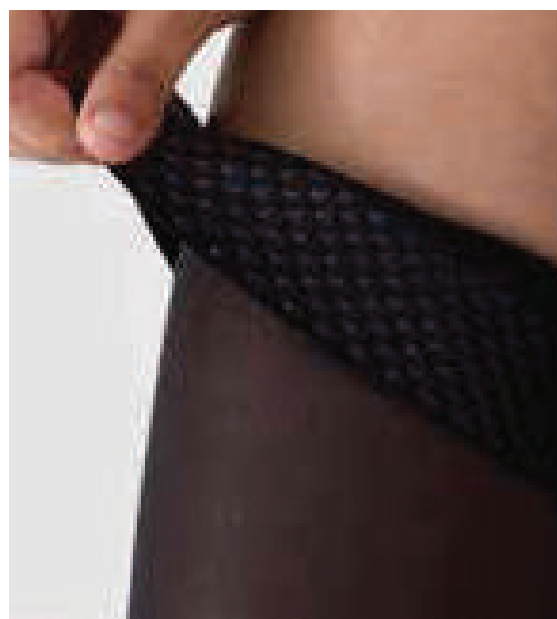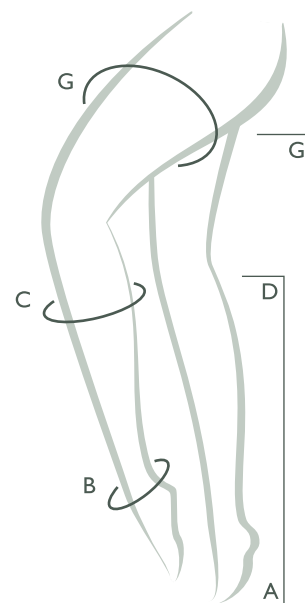

# FASHION INNOVA

## CHARACTERISTICS

The VARISAN® FASHION INNOVA knee-high stocking is RAL certified and available in 4 sizes and 2 lengths.

It has the appearance of a fashion product but the same characteristics as a class II medical stocking, with cotton in contact with the skin.

Its pleasing appearance overcomes the reluctance of many patients to wear a beige stocking. The microfibre yarn gives the stocking excellent breathability.

## INDICATIONS

CLASS II (23 - 32 mmHg)

- Superficial vein thrombosis, acute deep vein thrombosis, lymphoedema, prevention of post-thrombotic syndrome.

## CONTRAINDICATIONS

Acute dermohypodermatitis, skin infections, peripheral obliterant arteriopathy, known contact allergy to elastic support appliances.

## DURATION OF TREATMENT

Compression therapy with VARISAN® medical compression stocking should only be given during the daytime. The stocking is to be put on in the morning, if possible before getting up, and taken off in the evening, before going to bed.

During rest, the compression exerted could prove to be excessive.

For short afternoon rests there is no need to remove the stocking.

We recommend as much walking as possible during the daytime.

## GUARANTEE

The high quality of the materials used to make the VARISAN® elastic compression support appliance and the technologies adopted in their production guarantee the product for 6 months starting from the first time it is worn, providing the user instructions are followed.



### Compression classes available

Class	Compression in mmHg
II	23 - 32



### Colours

862 black









**TELEPHONE**

+61 438 806 337

**EMAIL**

[sales@ecsconsulting.com.au](mailto:sales@ecsconsulting.com.au)

**POST**

PO Box 8014, Wattle Park Vic 3128

**WEB**

[lymphoedemasc.com.au](http://lymphoedemasc.com.au)